

Easy Testing Steps:

Blood glucose test:



- 1 Insert test strip
"Thank you for using this product."



- 2 Apply sample
"Please apply blood into the strip."



- 3 Blood glucose data displayed
"The blood glucose is ninety eight milligram per deciliter."



- 4 Eject used strip

Specifications:

Model Number	FORA COMFORT plus voice V30
Measurement Range	20 ~ 600 mg/dL (1.1~33.3 mmol/L)
Test Strip	FORA COMFORT plus voice V30 test strips
Sample Volume	0.5 µL
Reaction Time	5 seconds
Coefficient of Variation	Less than 5%
Memory Capacity	450 results with date & time
Dimensions	96 x 50 x 23 mm
Weight	52 g (batteries excluded)
Power Source	Two AAA Alkaline Batteries
Data Transmission	Standard RS232 PC Interface

ForaCare Suisse AG
www.foracare.ch

Neugasse 55
9000 St. Gallen, Switzerland
Tel:+41-(0)71-220-1001 Fax:+41-(0)71-220-1075

FORA®



COMFORT plus voice V30

BLOOD GLUCOSE MONITORING SYSTEM



Talking- Voice Guided



Strip Ejection



AC/PC in Memory

Diabetes & Self Monitoring



Living with diabetes requires a healthy diet, weight control, and regular exercise, which benefits physical fitness and reduces the risk of:

- Heart disease
- Stroke
- Eye disease
- Nerve damage
- Kidney problems
- Gum disease and loss of teeth

A healthy lifestyle along with careful management of your blood glucose level is important for people with or without diabetes. Please consult medical professionals for further information on your blood glucose level.

Reference: National Diabetes Education Program. www.ndep.nih.gov

FORA®

COMFORT plus voice V30

BLOOD GLUCOSE MONITORING SYSTEM



Talking function

Talking- Voice Guided



No code setting required

No-Coding



4 sets

Daily Alarm



Avoid infection

Strip Ejection



Pre & Post-meal recording

AC/PC in Memory



Data transmission for better management


PC Link



AST

Less Pain with AST (Alternative Site Testing)

Blood sample can be obtained from palms, forearms, upper arms, calves, and thighs -- with proven equivalent accuracy to fingertip testing, thus reducing the pain on fingertips.

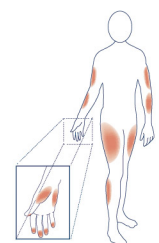

Talking meter makes self-testing easier



Talking

FORA's talking function makes blood glucose testing easy by guiding users through each step, and giving an audible readout of results.

Data Management

Testing data management is important for controlling blood glucose. FORA TeleHealth System assists users to transfer, organize and analyze data efficiently and wisely.



RS232/USB



Please learn more from www.foracare.com

