FORA® W310b

Digital Weight Scale

Digitale Personenwaage

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Bilancia Pesapersona Digitale

Báscula digital

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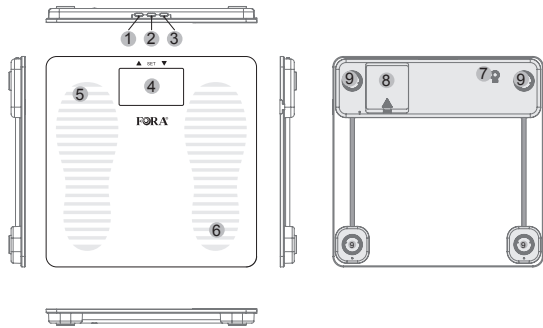
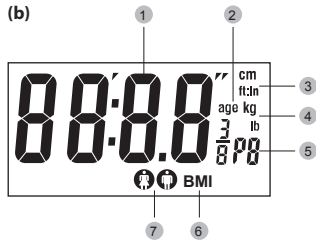
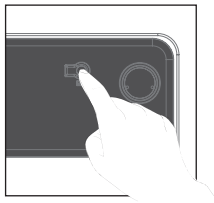
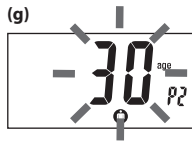
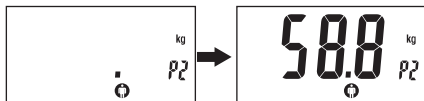
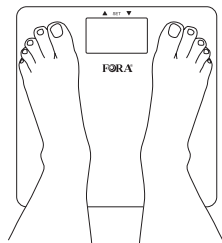
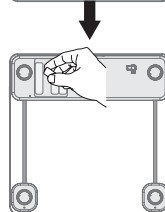
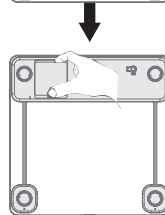
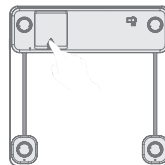
(o)

IMPORTANT SAFETY PRECAUTIONS

Thank you for purchasing the FORA W310b Digital Weight Scale. This instruction manual provides important information which helps you use this device. If you have any other questions regarding this product, please contact the place of purchase.

1. Use this device ONLY for the intended use described in this manual.
2. Read all the instructions and information in this manual before using this device.
3. Do NOT use the device if it is not working properly or damaged.
4. Consult your physician or healthcare provider before beginning a weight loss or exercise program.
5. Consult your physician before using this device if you are pregnant.
6. Do NOT step on the edge of the weighing platform or jump on the weighing platform.
7. Do NOT step on the weighing platform when your body or feet are wet.
8. Do not use the device on slippery surfaces such as wet floors.
9. Do NOT use microwaves or other devices that generate strong electrical or electromagnetic fields near the device. This may result in an operational failure.
10. Do NOT disassemble or try to modify the device.

KEEP THESE INSTRUCTIONS IN A SAFE PLACE

BEFORE YOU BEGIN

Body Mass Index

The Body Mass Index (BMI) is a number that is often called upon to evaluate body weight.

$$\text{BMI} = \frac{\text{body weight (Kg)}}{(\text{height (m)})^2}$$

$$\text{BMI} = \frac{\text{body weight (lb) x 703}}{(\text{height (in)})^2}$$

Recommended Measurement Time

To obtain an accurate reading, we recommend that you always take the measurement in the same environment and at the same time of day. Avoid taking measurements under the following conditions:

- Immediately after a bath or exercise
- After drinking alcohol or a large amount of water
- Less than 2 hours after a meal

If you take a measurement under any of the above conditions, the calculated body composition may be significantly different from the actual composition due to changes in water content in the body.

Contents of the System

Your new FORA W310b Digital Weight Scale kit contains:

1. Weight Scale
2. Owner's Manual
3. Warranty Card
4. Four 1.5V AAA Alkaline Batteries

Product Overview (a)

1. UP Button (▲)
2. SET Button
3. DOWN Button (▼)
4. LCD Screen
5. Toe Position
6. Heel Position
7. Weight Unit Button
8. Battery Cover / Compartment
9. Supporting Stand

Screen Display (b)

1. Measurement Results
2. Age Indicator
3. Height Unit
4. Weight Unit
5. Personal Profile Number
6. BMI Indicator
7. Gender

Screen Display (m)

1. Your Weight
2. Personal Profile Number
3. Gender: male
4. Your BMI
5. BMI Symbol

SETTING THE SCALE

Before using the scale for the first time, input these settings. To ensure accuracy, make sure you complete the steps below and save the desired settings.

► Setting the Unit of Weight

The unit of weight can be set to lb (pounds) or kg (kilograms). You can select the unit of weight by pressing the unit button to the desired unit indication. The function of this unit selection only can be used under setting and measuring mode. **(c)**

When the weight unit is set to pounds, the unit of height will be automatically set to feet and inches. When the weight unit is set to kilograms, the unit of height will be automatically set to centimeters.

► Adjusting Scale Settings

Start with the scale off. Press **SET** to turn the scale on to enter measuring mode. Press **SET** again to enter setting mode.

1. Select the personal profile number.

Press ▲ or ▼ to select a number from P1 to P5. Press **SET**. **(d)**

2. Enter the personal profile data.

Press ▲ or ▼ to select the gender of male or female. Press **SET**. **(e)**

Press ▲ or ▼ to set the height between 100 cm and 220 cm / 3'3"_{1/2} and 7'2"_{1/2}. Press **SET**. **(f)**

Press ▲ or ▼ to set the age between 6 and 99. Press **SET**. **(g)**

3. Press **SET** to exit the setting mode.

If you make a mistake before you have finished setting, repeat the steps from "Select the personal profile number".

TAKING MEASUREMENTS

Before Measuring

- Place the scale on a hard, flat surface. Soft or uneven flooring (e.g. carpets, rugs) may cause the scale to give an inaccurate weight reading.
- Take the measurement with dry feet, and position your feet correctly on the electrodes. Remove your shoes and socks before stepping onto the scale.
- Step onto the scale with your feet parallel to each other and your weight evenly distributed.
- Weigh yourself on the same scale at the same time every day, before breakfast or before getting dressed.

Measuring Body Mass Index (BMI)

1. Press **SET** to turn on the scale. The scale displays the profile setting of the last user. **(h)**
Press **▲** or **▼** to select the number. **(i)**

Note:

The scale will automatically turn on while stepping on the weighing scale under standby mode.

2. The scale displays the profile setting of the last user. Wait for "0.0" to appear on the **LCD**. **(j)**
3. Step onto the weighing platform.
Stand still and distribute your weight evenly while the scale is measuring your weight. **(k)**

Note:

The scale will automatically turn off if the weight is unstable for more than 30 seconds.

When the value is stabilized, it will flash three times and the scale will then calculate your BMI. **(l)**

4. Display the Result. The scale shows your measurement results as follows. **(m)**
5. Step off the weighing platform. The result is saved in the scale memory and enter the transmission mode (PCL).

SCALE MEMORY

Your scale stores measurement results with the corresponding personal profile number in the scale memory.

Results in memory can be transmitted to the mobile device by wireless connection.

This allows you to transmit and review your measurement data on a mobile device for up to 135 sets of measurement results.

BLUETOOTH PAIRING AND DATA TRANSMISSION

You can transmit your data from the meter to your devices (e.g. smart phone, tablet, PC...) via Bluetooth. Please contact your local customer service or place of purchase for assistance. Please note that you must complete the pairing between the meter and Bluetooth receiver before transmitting data.

► Pairing with your mobile device

1. Turn on the Bluetooth function on your mobile device.

2. Long press SET on the weight scale until PCL displayed on the screen during the pairing process.
3. On your mobile device, launch the iFORA WS app.
4. Select the Setting label and then tap the Pairing button.
5. Tap the Search button to search for available BLE meters. Wait for a few seconds, a pop-up window shows up to display the search results.
6. Select "FORA W310" to start pairing.
7. When pairing is complete, "FORA W310" will be displayed in the Available BLE Meters field on your device.
8. Tap  to pop up the message window. Select Yes to save the setting.

► Data transmission

After each measurement, the Bluetooth will be automatically activated and the weight scale will try to connect to the paired device (e.g. smart phone) to transmit the data.

► Mobile phone compatibility issues

The Bluetooth functionality is implemented in different ways by the various mobile phone manufacturers.

Unfortunately, in some mobile phone models, even with Bluetooth functionality, they may be compatible only with certain types of devices. If a problem occurs in the connection between your mobile phone and the weight scale, or if you are uncertain regarding your mobile phone's Bluetooth capabilities, please consult your mobile phone manual or contact your local customer service for assistance.

Note:

- While the weight scale is in transmission mode, it will be unable to perform a test. Press SET to return to measuring mode.

- Make sure your device has the Bluetooth function turned on and the weight scale is within the receiving range before transmitting the data.

MAINTENANCE

Your scale takes four 1.5V AAA alkaline batteries.

Low Battery Signal

The scale will display the message shown to alert you when the scale power is running low. This means it is about time to replace the battery. **(n)**

WARNING!

If the incorrect type of battery is used, it may cause damage to the scale.

Replacing the Battery

When replacing the battery, make sure that the scale is turned off.

1. Press the edge of the battery cover and lift it up.
2. Remove the old batteries and replace them with four 1.5V alkaline batteries
3. Close the battery cover. **(o)**

Note:

- Replacing the battery does not affect the test results stored in memory.
- Properly dispose of the batteries according to your local environmental regulations.
- Batteries may leak chemicals if left unused for a long time. Remove the batteries if you are not going to use the device for

an extended period (i.e. 3 months or more).

Cleaning

1. To clean the outside of the scale, wipe it with a slightly damp cloth and water or a mild cleaning agent, then dry the device with a soft dry cloth. Do **NOT** immerse the device in water.
2. Do **NOT** use chemical or organic solvents to clean the scale.
3. Wipe the scale clean at once if any grease, sauce, vinegar, juice or other substances that may cause damage are spilt on the scale.

Scale Storage

1. Storage condition: -25°C to 70°C (-13°F to 158°F), below 93% relative humidity.
2. Avoid dropping the scale or heavy impact.
3. Avoid direct sunlight and high levels of humidity.

SYSTEM TROUBLESHOOTING

Error Messages

MESSAGE	WHAT IT MEANS	WHAT TO DO
	The weighing subject exceeds the maximum capacity of the device.	Remove the weighing subject from the platform to avoid damage.

If you follow the recommended action but the problem persists,

or if error messages other than the ones shown above appear, please contact local customer service. Do not attempt to repair the device by yourself and do not try to disassemble the scale under any circumstances.

SYMBOL INFORMATION

Symbol	Referent
	Manufacturer
	Consult instructions for use
	Serial number
	Type BF Equipment
	Collection for electrical and electronic equipment
	Caution, consult accompanying documents
	Temperature limitation
	Humidity limitation
	CE mark

SPECIFICATIONS

General Information	
Model no.	FORA W310b
Power source	Four 1.5V AAA alkaline batteries
Dimensions & Weight	300 (L) x 300 mm (W) x 25 mm (H), 1600 g without batteries
Size of LCD	91 (L) x 52 (W) (mm)
LCD	Backlight LCD
Range	Gender: Male / Female Height: 100 to 220 cm / 3' 3" ^{1/2} to 7' 2" ^{1/2} Age: 6 to 99 Weight: 3 to 180 kg / 6.6 to 396 lb
Operating conditions	5 °C to 40 °C (41°F to 104°F), below 85% R.H.
Storage conditions	-25°C to 70°C (-13°F to 158°F), below 93% R.H.
Battery life	At least 180 measurements (3 minutes each time) or at least 6 months on standby.

User profile	5 individuals
Weight unit	lb / kg
Transmission	Bluetooth

This device has been tested to meet the electrical and safety requirements of: IEC/EN 60601-1, IEC/EN 60601-1-2, EN 301 489-17, EN 300 328

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**ForaCare Suisse AG**